Medical Devices

Pressure gauge kit

IT > MANUALE ISTRUZIONI D'USO
EN > OPERATING INSTRUCTIONS MANUAL
DE > BEDIENUNGSANLEITUNG
FR > MODE D'EMPLOI
NL > GEBRUIKSAANWIJZING

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Classe di accuratezza: Kl.1,6

Ugello calibrato: 0,53 mm

The Pressure Gauge Kit is used to measure the operating pressure of a compressor, thus verifying that the device is functioning correctly. It must be used exclusively with FLAEM compressors.

DANGER WARNINGS: • It is forbidden to disassemble or tamper with the device. Some components of the device are so small that they could be swallowed by children; therefore keep the device out of the reach of children. • Do not use the connection tube (if provided) outside its intended use, as it could cause a strangulation hazard.

GENERAL WARNINGS: • Handle with care. Accidental impacts or drops could compromise proper operation and measurement accuracy. • Use in dusty environments may compromise the proper functioning of the device. • If your device stops working, contact FLAEM NUOVA S.p.A. • warning attention: The device may be used by centers or service centers authorized by Flaem Nuova. The device must NOT be used by the end user.

INSTRUCTIONS FOR USE. • Insert the connection tube (B1) onto the appropriate connector of the pressure gauge (M1), connect the other end of the connection tube to the compressor air outlet (B2.). • Turn on the compressor, after approximately 30 seconds read the pressure value on the pressure gauge. Depending on the type of compressor, verify that the reading is within the “correct pressure” range indicated in the table below.

	Pressure Gauge Value (M)
All Flaem models from List A	0,90 – 1,15 bar
All Flaem models from List B	1,15 – 1,45 bar

COMPRESSOR CHECK INTERVAL. The accuracy of the pressure generated by the compressor must be verified every 100 hours of operation or at least once a year.

PRESSURE GAUGE KIT CHECK AND COMPONENT REPLACEMENT INTERVAL. • The accuracy of the pressure gauge must be verified once every two years. • Calibration and verification of the pressure gauge must be carried out by an authorized service center. • If calibration is not possible, the kit must be replaced. • The connecting tube must be replaced once a year.

CLEANING INSTRUCTIONS. Use only a cloth dampened with an antibacterial detergent (non-abrasive and free of any solvents).

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© 2026 FLAEM NUOVA®